

THE
LISTENING
 PROGRAM®



- 1 Connect the provided 6" cable from the headphone port on your device (iListen® or portable CD player*) to the IN port on the headphone amplifier
- 2 Connect the mono headphone cable to the BC port on the amplifier
- 3 Connect the stereo headphone cable (blue ring at end) to the AC port on the amplifier
- 4 Power amplifier ON/OFF by pressing and holding power button until you see "powering on" or "powering off" on the display

** Your amplifier is designed to only be used with an iListen® or a portable CD player such as the Sony Walkman/Discman. Use of another device may damage the amplifier, due to power incompatibility.*



The headphone amplifier uses AA alkaline batteries which last for up to 10 hours. Power amplifier OFF when not in use to conserve battery life. Do not use rechargeable batteries.

Adjusting Volume

The amplifier allows you to adjust air conduction (AC) volume, which is what you hear through the ear cups, and bone conduction (BC) volume, the amount of vibration coming through the bone conductor.

Always make sure the volume on your device (iListen® or portable CD player) is set at the maximum volume to ensure adequate power is going to the amplifier. Volume levels are preset on the amplifier at 5 on a loudness scale of 0-10, with 0 being silent and 10 maximum volume.

When you power your amplifier on, you will see AC and BC on the Main menu. To select the AC or BC menu, highlight the desired menu using the arrow keys and press select. Then adjust volume to desired setting and return to the Main menu. The amplifier will save your settings until you change them.

If you need to make volume adjustments for more comfortable listening, adjust the AC volume on the amplifier for desired comfort level. Do not adjust for comfort on your iListen or CD player, as this will reduce the power needed for bone conduction. Do not change the channel balance or BC volume unless instructed by your provider.

Helpful Tips

- 1 Turn amplifier off when not in use
- 2 Change batteries before 10 hours of use
- 3 Make sure all connections are secure; amplifier, ear cups, and devices
- 4 Remove the protective film on amplifier display for better screen visibility
- 5 If you hear static, check connections to ensure tightness, and/or replace batteries as needed